




The World's Most Advanced Massage Chairs™



68 Route 125
Kingston, New Hampshire 03848
U.S.A.
T. (603) 347-6006 F.(603) 642-9291
www.infinitetherapeutics.com

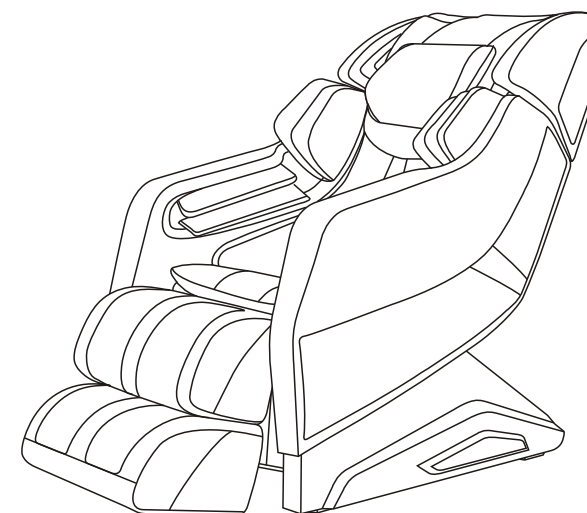

Deluxe Multi-functional Massage Chair

Riage
USER MANUAL

Thank you for your purchase of this massage chair. Before use, please read this USER MANUAL carefully, and pay special attention to the safety precaution to ensure proper operation and use.

Please keep this USER MANUAL properly for further reference.

▶ CONTENTS

SAFETY PRECAUTIONS	1
NAME AND FUNCTION OF EACH COMPONENT	3
External structure	3
Internal structure	3
Power box	4
Controller	5
Controller buttons	6
LCD screen icon decomposition	8
FUNCTIONS AND FRATURES	10
PRECAUTION BEFORE USING	11
Installation Method	11
Move method	15
Turn on power supply	15
Installation position	16
Floor protection	16
Massage chair commissioning	17
Before sitting in the chair	17
After using	18
Surrounding environment	18
Grounding	18
PRODUCT STORAGE AND MAINTENANCE	19
TROUBLE SHOOTING	20
TECHNICAL SPECIFICATION	21

▶ TECHNICAL SPECIFICATIONS

Name: Deluxe Multi-functional Massage Chair

Model: Riage

Rated voltage: 100-120V~

Rated frequency: 50/60 Hz

Rated power: 240W

Safety design: Class I

▶ TROUBLE SHOOTING

No.	Problem	Possible Cause	Solution
1	The sound “Xu Xu”, “Ta Ta” is heard.	It is a normal sound made by the working motor or the sound is produced when the massage ball is rubbed against the cloth cover.	It is normal and no need for treatment.
2	The machine can not be started up.	The power supply plug is detached.	Insert the power supply plug into the socket again.
		The power supply switch is not turned on.	Turn on the switch
		The fuse of massage chair blew out.	Replace a new one with the same mode
3	The airbag installed at seat cushion or legrest can not work.	To check whether there is air tube blockage.	To clear the air tube

If abnormal phenomenon still remains after above measures have been taken to the massage chair, please contact local distributor for repair and maintenance.

Note: Do not dismantle the product by yourself, and our company shall shoulder no responsibility to the damage of the product or the injury of people caused by improper use! Thanks your cooperation.

▶ SAFETY PRECAUTIONS

IMPORTANT SAFETY INSTRUCTIONS

When using the massage chair, basic precautions should always be followed, including the following: Read all instructions before using this massage chair:

DANGER. To reduce the risk of electric shock:

1) Always unplug this massage chair from the electrical outlet immediately after using and before cleaning.

WARNING. To reduce the risk of burns, fire, electric shock, or injury to persons:




- 1) A massage chair should never be left unattended when plugged in. Unplug from outlet when it is not in use, and before putting on or taking off parts.
- 2) Do not operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to persons.
- 3) Close supervision is necessary when this massage chair is used by, on, or near children, invalids, or disabled persons. Keep children away from reclining backrest and extended footrest.
- 4) Use this massage chair only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- 5) Never operate this massage chair if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the massage chair to service center for examination and repair.
- 6) Do not carry this massage chair by supply cord or use cord as a handle.
- 7) Keep the cord away from heated surfaces.
- 8) Never operate the massage chair with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- 9) Never drop or insert any object into any opening.
- 10) Do not use outdoors.
- 11) Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- 12) To disconnect, turn all controls to the off position, then remove plug from outlet.
- 13) Do not stand on or in the massage chair. Use only while seated.
- 14) Use heated surfaces carefully. May cause serious burns. Do not use over insensitve skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous
- 15) Connect this appliance to a properly grounded outlet only.see Grounding Instructions.

SAVE THESE INSTRUCTIONS

THIS MASSAGE CHAIR IS INTENDED FOR HOUSEHOLD USE ONLY

SAFETY PRECAUTIONS

CAUTION

 COMPULSARY	<ul style="list-style-type: none"> ◆ Please do not continuously use the massage chair over 20 minutes, and it is limited to massage a particular part of body in 5 minutes. ◆ Please put the massage chair on flat ground for use. ◆ When adjusting the legrest or backrest, please do not exert excessive force on it; in addition, keep children and pets well clear. ◆ When removing power plug, please hold the insulation plug, do not pull power cord. ◆ Please do not use the power supply that is beyond the specified electric voltage, as this may cause fire disaster or damage massage chair. ◆ Before using, please uncover the back cushion to check whether backrest or fabric is damaged; if damaged, please stop using this massage chair immediately, and contact franchiser for repair immediately. ◆ Please do not operate the controller with damp hands, and do not put heavy objects on controller. ◆ Only people less than 120kgs are acceptable. ◆ Please do not use other therapeutic equipments together with this massage chair. ◆ During use, if you feel uncomfortable, please stop use immediately and consult your doctor. ◆ This massage chair is designed and intended for non commercial household use only, do not use the massage chair for medical treatment. ◆ The massage chair is equipped with heating function, so people who are insensitive to heat must use it carefully. ◆ Always switch off power and remove the plug from the mains socket immediately after use and before cleaning.
 PROHIBITED	<ul style="list-style-type: none"> ◆ To avoid causing discomfort, do not use the massage chair immediately after meals. ◆ Do not fall into sleep on massage chair, when the massage chair is working. ◆ Do not use the massage chair when you are drunk or feel bad. ◆ If the upholstery becomes dirty, please use only known good quality leather cleaner, do not use chemical cleaners, petrol or detergents etc, and do not spray pesticide on it. ◆ Do not sit on backrest, legrest, or armrest to avoid accidental injury or machine fault. ◆ Please do not smoke while using the massage chair.
 REMOVE PLUG	<ul style="list-style-type: none"> ◆ Before maintenance, please pull out power plug. In addition, please do not insert or unplug the power plug with damp hands, to avoid electric shock. ◆ When the massage chair is not in use, please switch off power and remove the plug from socket. When power is off, please cut off power supply immediately and pull out the power plug from socket.

PRODUCT STORAGE AND MAINTENANCE

Maintenance

- ▶ Please twist the cotton cloth which has been dipped in neutral detergent before wiping the back cover, hand rest, legrest, air tube and some other plastic components, then use a dry cotton cloth to wipe them again.
- ▶ Use dry cloth to wipe the controller, DVD, and power supply box.
- ▶ Use a piece of cloth that has been dipped with neutral detergent to wipe the back cushion and pillow, then use a piece of wet cloth to wipe them, and finally put them to a well-ventilated place for natural drying.

Caution

- ▶ Make sure to pull out the power supply plug firstly before maintenance. Do not use wet hands to pull out the plug to avoid electric shock. When clean the back cover, please be cautious of detergent entering into the power supply box!
- ▶ Do not iron the upholstery.

Storage method

- ▶ Wipe off dust and dirt from the product after use, if the product is not to be used for a long period of time, please cover the product with a dust cover (self provided).

Caution

- ▶ Do not store the product under direct sunlight or in places of high temperature. This may cause the color of the upholstery to fade.

Kind reminding

During daily use, please check whether the following symptoms exist:


- ▶ Scorched smell exists.
- ▶ When touch the wire, the power is on, but sometimes off.
- ▶ The power supply cord is hot.
- ▶ Other abnormal symptom

Caution

- ▶ Please stop using the massage chair to avoid breakdowns and accidents occurring if any above symptom exists.
- ▶ Turn off the power supply switch and pull out the power supply plug, and then contact the local distributors or headquarters for repair and maintenance.
- ▶ Except all the normal operation and maintenance mentioned above, any other repair and maintenance can only be operated by our authorized dealer or service agent.

▶ NAME AND FUNCTION OF EACH COMPONENT

Power box

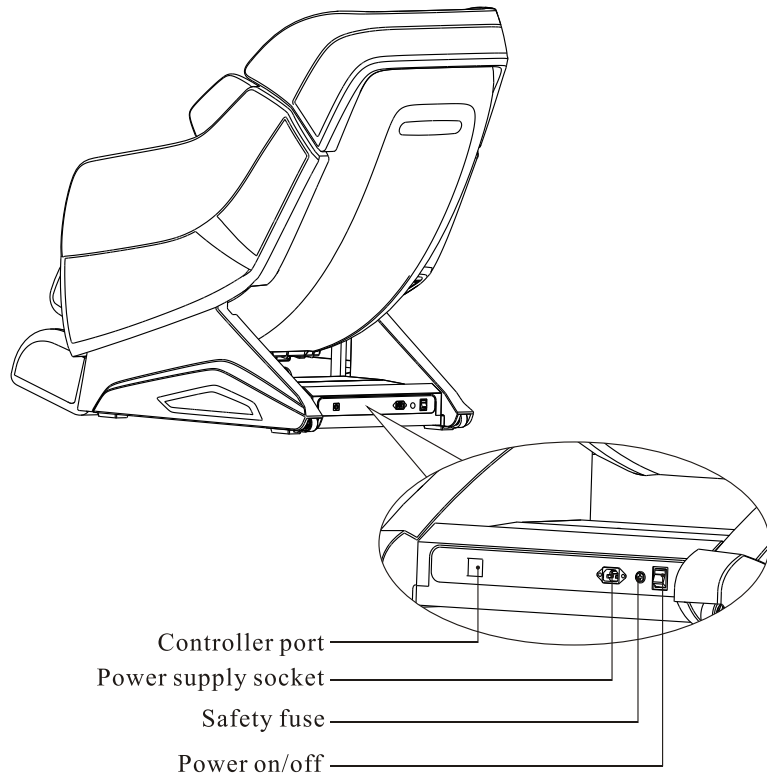


⚠ Caution

- ▶ Do not let children play on the chair in case that accident happens due to the power supply box.
- ▶ Please power off the chair and unplug the chair from the power supply socket after the massage finishes
- ▶ Do not use the chair under thunders in rainy days.
- ▶ Please power off the chair and unplug the chair from the power supply socket when the chair is not in use, so that children won't accidentally get the chair started.
- ▶ Please store the chair if it is not in use for a long time.

▶ PREPARATION BEFORE USING

Massage chair commissioning

- ▶ Before commissioning, please make sure there is enough space around the chair. Then use the controller to recline down the chair backrest to the limit position, rise up the legrest to the highest position and stretch out to the limit length.
- ▶ Testing all functions work normally, then power off the chair, return back to original position.
(How to select the program, please refer to “ Controller instruction” and “display screen menu instruction”)

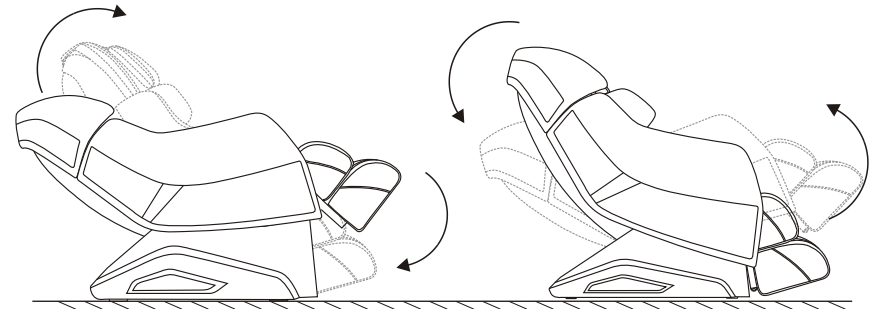

⚠ Caution

- ▶ Do not put fingers or any other objects into the gap between the seat and legrest.
- ▶ When adjusting up and down angles of the backrest and legrest, please make sure there is no obstacle around the massage chair.
- ▶ During using, please make sure the power cord is not entangled or clamped but laid orderly.

Before sitting in the chair

- ▶ Make sure the legrest is well installed.
- ▶ Make sure the chair is in original position, if any part is not in initial position, please power on/off again to make the whole chair go back to original position.

⚠ Caution


- ▶ If the legrest rises up, please do not seat on the chair.
- ▶ Do not make the legrest to suffer from heavy loading suddenly, or it may damage the legrest mechanism.

► PREPARATION BEFORE USING

Installation position


Safety distance for the periphery of massage chair.

- Please make sure there is adequate space for installation this product:
 - Back: minimum 60cm from the wall or any other blocks;
 - Front: minimum 30cm from any blocks;
- In order to prevent from signal interference, please keep at least 1m away from the TV, radio and other video and audio devices.

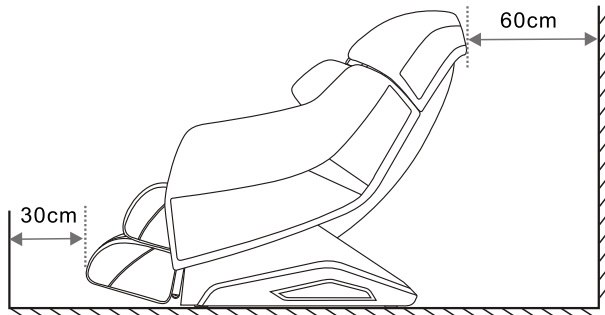


Floor protection

- Please put carpet or other pad under the massage chair, in order to avoid any unpredictable impression on the floor.

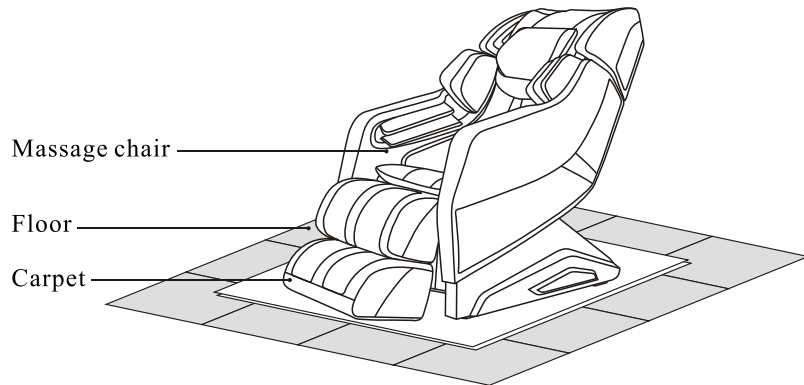

► NAME AND FUNCTION OF EACH COMPONENT

Controller

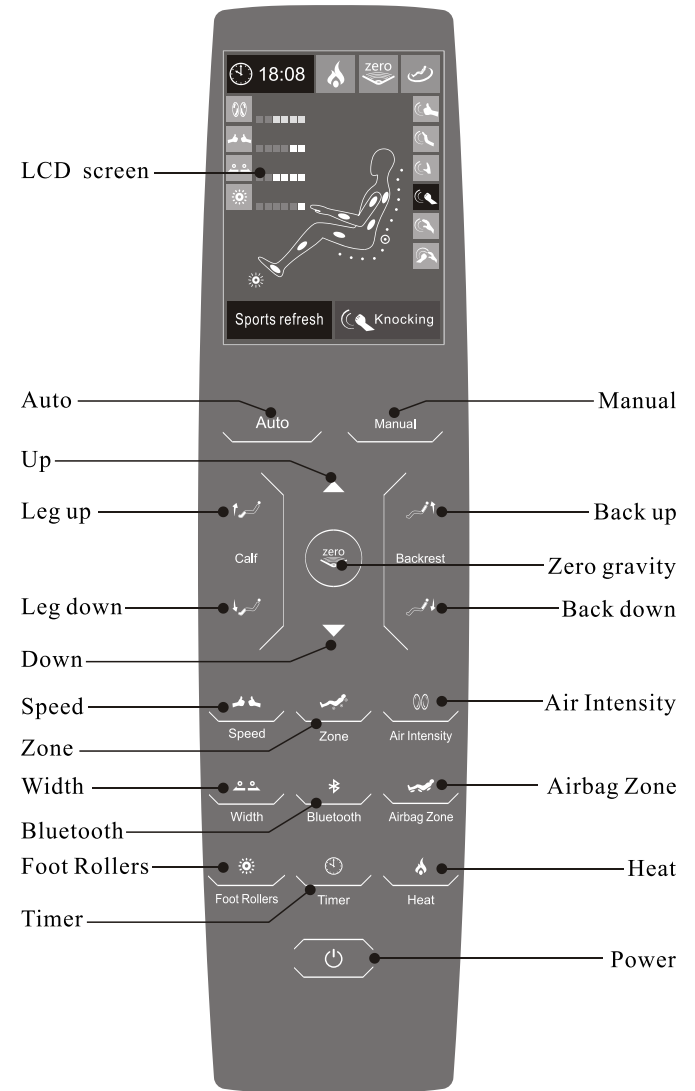

▶ NAME AND FUNCTION OF EACH COMPONENT

Controller Buttons



- Power:** to start/end massage function of the chair
- ▶ With electricity connection, press this button to initiate main menu and the LCD is on. You can choose Auto, Manual, Airbag, Heating, Roller massage function for setting.





- Auto**
- ▶ Press this button to enter Auto menu. You can choose Extension, Working relief, Sports refresh, Rest & Sleep, Neck & Shoulder, Waist & Spine or AirPressure massage programs according to your own body state and massage time. (See the LCD menu for details)



- Manual**
- ▶ Press this button to enter into Manual programs. You can choose Shiatsu, Rubbing, Tapping, Knocking, Kneading, Sync massage mode according to your own body state and massage time. (See the LCD menu for details)





- Leg up and down**
- ▶ Keep pressing “” , the leg rest raises to the highest; stop pressing, the leg rest stays.
 - ▶ Keep pressing “” ,the leg rest descends to the lowest; stop pressing, the leg rest stays.





- Zero Gravity**
- ▶ Press this button to start(end) zero gravity function, the icon is on in the LCD screen; when you manually adjust the backrest and leg rest, if the icon is off, it is zero gravity state.
 - ▶ Zero gravity creates a completely relaxing, naturally comfortable massage position; Somatological design makes back, buttocks and leg stay in a certain angle like lying in space capsule, under which the body pressure is thoroughly released and the muscle come into.



- Shoulder Position adjustment**
- ▶ When it is in manual mode, keep pressing “” or “” to adjust massage point.
 - ▶ The buttons are also for shoulder Position adjustment.




- Backrest up and down**
- ▶ Keep pressing “” ,the backrest raises to the highest; stop pressing, the backrest stays.
 - ▶ Keep pressing “” ,the backrest descends to the lowest; stop pressing, the leg rest stays.

▶ PREPARATION BEFORE USING

Move method

- ▶ There are moving rollers at the bottom of the chair.
- ▶ Raise the back to the highest point.
- ▶ Switch off and plug out the chair.
- ▶ Lift the front part of the chair as showed in the figure(one person raise the front part via holding moving holder, another person push the chair when holding the top of the chair) , move the chair by the rollers.

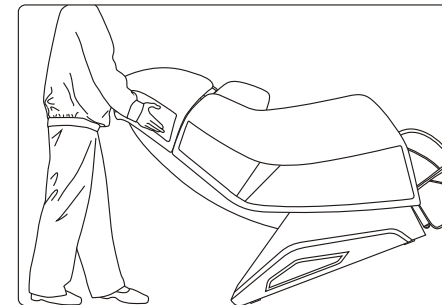

Caution

- ▶ Don't move it when there is people in it.
- ▶ Don't use the armrest while moving the chair, only hold the moving holder in the back and front

Turn on power supply


- ▶ Plug the power cord into three core socket with well grounding.
- ▶ Tun on the power switch on the bottom of massage chair.

Caution

- ▶ Before switching on the chair, please make sure there is no damage on the power wire and other exposed wires.
- ▶ Make sure the power switch is off before the chair is connected to electricity.


PREPARATION BEFORE USING

- ▶ Lift the legrest unit, and connect the connectors of legrest to the relevant ports on the front frame of the seat frame well. (see diagram below)
- ▶ Connect the Pu covers of the legrest unit to the seat with zipper.

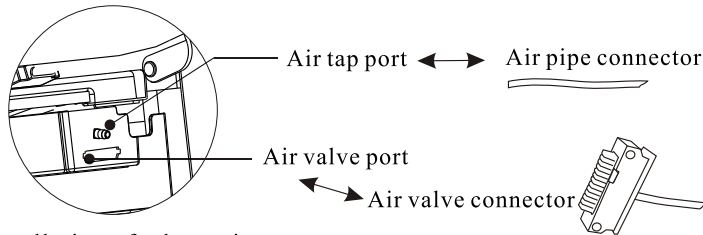

Step 4 Installation of other units

- ▶ Insert the plug of remote control into the socket on the power box at the bottom.
- ▶ Connect the power cord connector to the power cord port on the power box. (see diagram below)

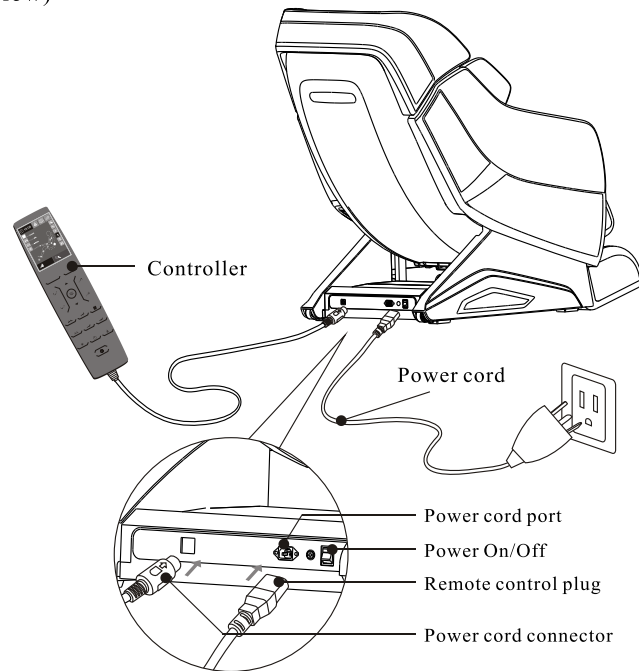

- ▶ After all parts assembled, please insert the power cord into the power socket, and then turn on the switch, press the power on the remote control to start massage. (see diagram above)

NAME AND FUNCTION OF EACH COMPONENT



Massage speed adjustment

- ▶ Under manual mode, press this button to choose 6 different degrees of massage speed, and there is indication on the LCD screen.



Zone massage

- ▶ Touch this button to select FULL, PARTIAL, POINT massage at your back.



Air Intensity adjustment

- ▶ Press this button to choose 5 different degrees of airbag intensity, which shows in LCD screen.



Width adjustment

- ▶ Under manual mode, if the light of button is on, it is ready for operation. Press this button to choose wide, medium or narrow massage range. Indicating light is on or off.
- ▶ When the light of button is off, this function is not available.



Bluetooth

- ▶ Touching this button to turn on/off the Bluetooth, the icon will be light up or go out.



Airbag Zone

- ▶ Press this button to start or end shoulder, leg, seat, whole body airbag massage. Indication on the LCD is on or off.



Foot Rollers massage adjustment

- ▶ Press this button to start or end foot rollers massage, you can choose high, medium or low modes. There is indication on the LCD screen.



Timer

- ▶ Press this button to choose timer for massage: 10 mins, 20mins or 30 mins.




Heating

- ▶ Press this button to start heating function on back light of icon on; press this button again to shut down this function with the light off.


▶ NAME AND FUNCTION OF EACH COMPONENT

LCD screen

Auto programs

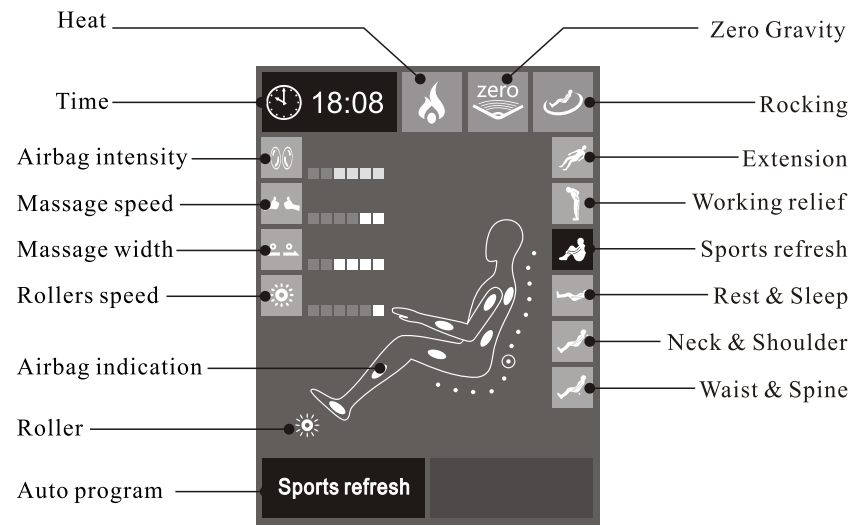


Manual programs

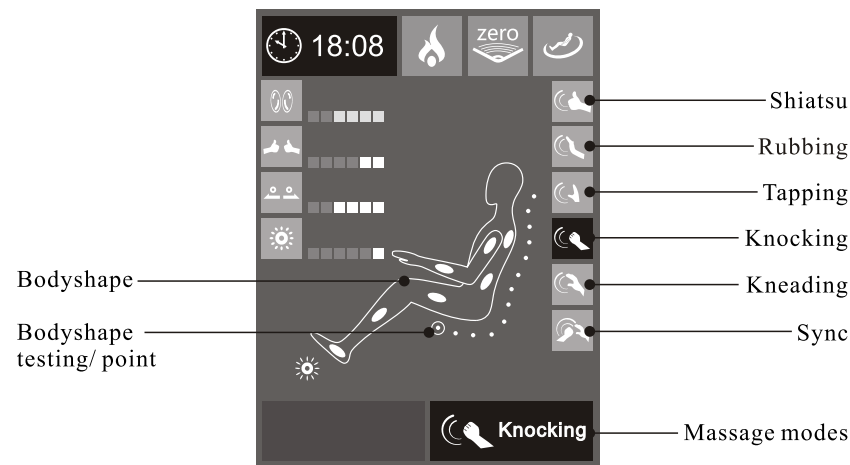


▶ PREPARATION BEFORE USING

Step 3 Installation of legrest unit

- ▶ Lift the calf part and connect the calf with the seat frame by hanging (pay attention to the balance), move the hanger to the axis on one side, then do the same for the other side.

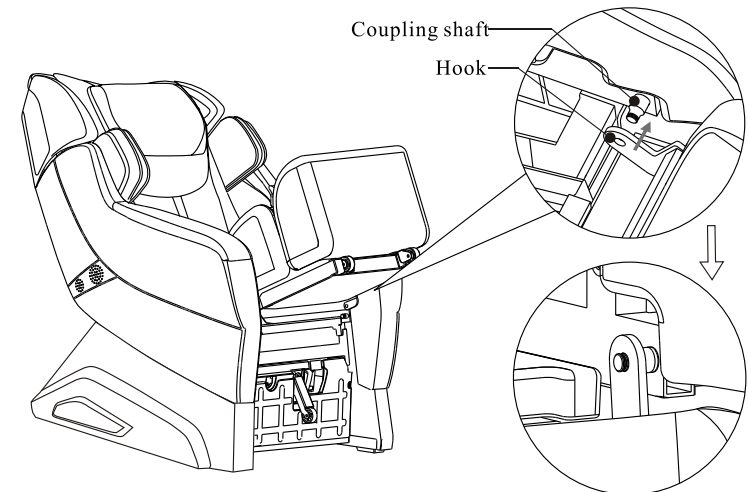


- ▶ After the calf is well connected with the axis, fasten it by the lock on the left and right side. (see the figure below)

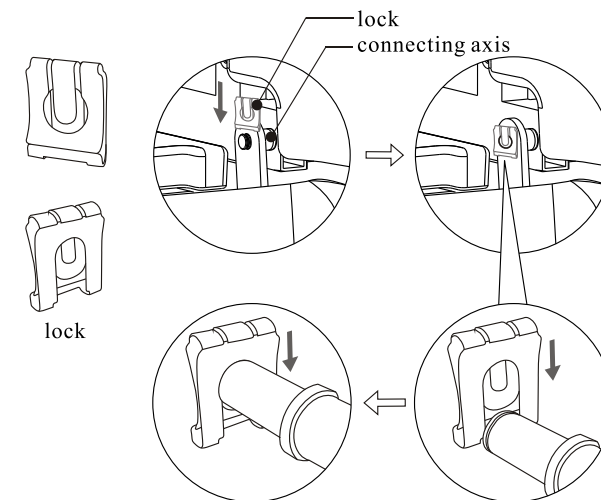

► PREPARATION BEFORE USING

- Hang the hook of the armrest well to the connecting rod of the backrest and then connect the armrest to the side of the seat frame, push it down to make it connected well. (Diagram 2)
- Fasten the armrest to the seat frame with one screw on each side. (Same assembly method for each side of armrest)

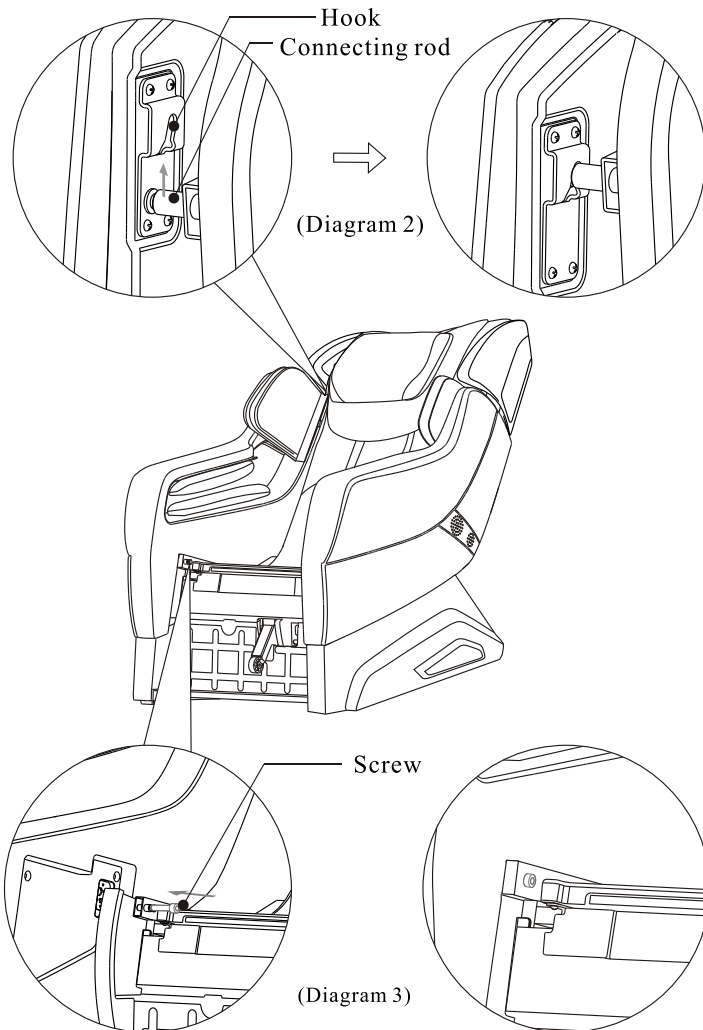

► NAME AND FUNCTION OF EACH COMPONENT

LCD Menu

► Auto programs

With electricity connected, press power button, the LCD is on, press Auto to enter the Auto program menu, you can choose Extension, Working Relief, Sports refresh, Rest & Sleep, Neck & Shoulder, Waist & Spine, Air Pressure programs. (see the pic below).

Extension:

This massage comes from Thai Massage, the strong kneading and stretch of the body can effectively stretch legs, waist and other parts, and it's the best choice to relax tired muscles and increase physical vigor.

Working Relief:

This is an exclusive massage program for the workplace walks, according to the features of long time desk work and business trip, it can ease muscles and recuperate body.

Sports refresh:

This is designed for the muscles after sports activities and exercise. It can ease and relax muscles after intense exercise, especially promote the body circulation system to improve the muscle recovery after exercise.

Rest & Sleep:

This is the caring program to help improve the quality of lunch break and sleep. The mild massage kneads body parts from heavy strength to light one, from light to no strength to help soothe tired body gradually.

Neck & Shoulder:


This massage is to relieve neck and back pain symptom. 80% of the massage time will focus on the shoulder and neck. The curve design of the chair backrest makes the massage rollers completely touch the shoulder and the neck, so it can effectively relieve your fatigue and stiffness of these parts.

Waist & Spine:


This massage is to relieve the symptoms of lumbar spine pain. 80% of the massage time will focus on the lumbar spine and the rollers will conduct specific area massage to relieve the sense of tension and keep the health of this part.

► Manual programs

With electricity connected, press power button, the LCD is on. Press Manual to enter Manual Program menu. Shiatsu, Rubbing, Tapping, Knocking, Kneading, Sync massage modes are for choice. (see the pic above)

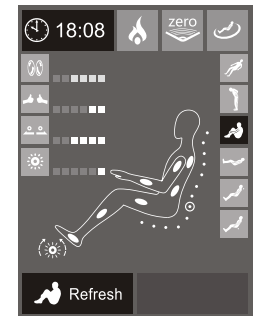

Auto programs interface

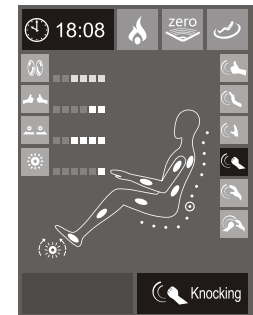

Manual programs interface

FUNCTIONS AND FRATURES

1.Audio Features

Product is designed with Bluetooth audio connection function.

2.Shoulder location

Shoulder location function gives the most effective massage for people of different height.

3.Zero Gravity

This chair reflects zero gravity principle from the space capsule seat of NASA, the theoretical basis of which is that the legs are higher than heart , it can imitate weightless status of aerospace, thus fully releasing the weight of body, then body will stay in most comfortable status without any stress.

4.Four wheel massage mechanism

The innovative four wheel massage mechanism and traveling structure perfectly match ergonomic human engineering.

5.Air pressure massage

Air pressure massage for back, waist, shoulder, arm, buttock, calf, and feet.

6.Roller massage for sole


Double row rollers massage on foot conduct an all-around shiatsu massage onto the sole.

7.Diversified massage modes

Diversified massage modes: knocking, kneading/knocking, kneading, shiatsu, which act as a professional massagist.

8.Swing function

Product features front and rear swing function that takes you into the fantasy space swinging state and a thorough relaxation.

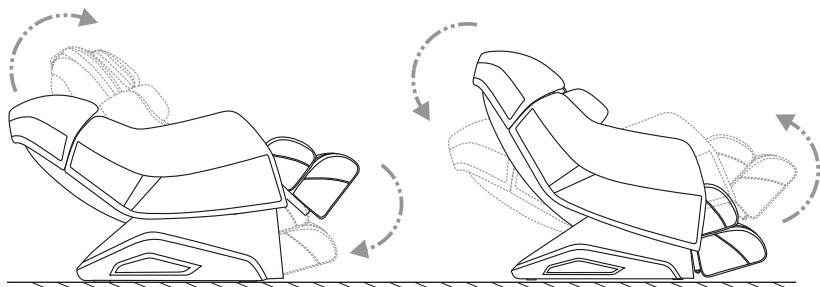

9.L shape super long massage track

PREPARATION BEFORE USING

Installation Method

Step 1: Get out each part of the chair

- ▶ Unpack the cartons, carefully get out sewing units, accessories box and so on.
- ▶ Check all accessories with packing list.

Step 2: Installation of armrest

- ▶ Connect the power code and remote control to the relevant socket on the power box, turn on the power and make the backrest to the original position. (follow the sequence in the diagram below)
- ▶ Match the air valve, air-pipe plugs on armrest to the side ports on lower part of the armrest frame. (diagram 1)

